

## Sample Submission Guide for the Molecular Detection of Select Zoonotic Pathogens

Pathogen	Sample type*						
	Whole Blood	CSF	Synovial Fluid	Wound Fluid/Pus	Urine	Tissue	Paraffin Block
<i>Anaplasma phagocytophilum</i>	✓	✓	×	×	×	×	×
<i>Babesia</i> species	✓	×	×	×	×	×	×
<i>Bartonella</i> species	✓	✓	✓	✓	×	✓	✓
<i>Borrelia burgdorferi</i> s.l	✓	✓	✓	×	×	✓	✓
<i>Ehrlichia chaffeensis</i>	✓	✓	×	×	×	×	×
<i>Leptospira</i> species	✓	✓	×	×	✓	×	×
Relapsing fever spirochetes	✓	✓	×	×	×	×	×
Minimum volume	Refer to pathogen GTS	0.5 mL			5 mL	1-5 mg	
Shipment conditions	Store specimens refrigerated for up to 5 days and ship with freezer packs. If stored > 5 days, freeze at -20 °C and ship on dry ice.						Ambient

\*Additional sample types may be tested upon consultation with the Field Studies Section.

April 2024, v2

✓ = Ideal

✓ = Acceptable

×

 = Unacceptable